
premierdentalco.com · Made in U.S.A. 888-670-6100 • 610-239-6000 Plymouth Meeting, PA 19462 U.S.A. 1710 Romano Drive, Premier® Dental Products Company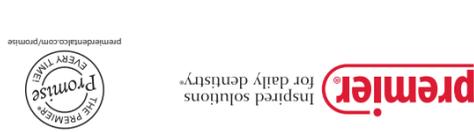

premierdentalco.com/premier-powerful-enzymatic-action

**premier®**

## Brite Shield™

### Instrument Cleaner and Protectant

premier® Brite Shield™ Instrument Cleaner and Protectant

### Brite Shield™ Instrument Cleaner and Protectant

#### Intended Use:

Brite Shield™ is a static soak and ultrasonic cleaner that chemo-mechanically debrides clogged dental instruments and aids to protect instruments through a unique passivation process to restrict rust and corrosion.

#### Device Description:

Premier® Brite Shield™ Power enzymatic cleaning action to remove proteins, light stains, set cements, and protects instruments from rust and corrosion.

#### Safe to be used:

- with stainless steel instruments and burrs.
- with metal trays,
- with diamond cutting tools
- without ultrasonic machine; soak time: 15-20 min.

#### Contraindications:

- Do not use Brite Shield™ under the following conditions:
- Do not use on anodized aluminum Instruments or Bur Blocks
- If the container or cap is found broken/damaged
- Should not be used on anodized aluminum instruments

#### PRECAUTIONS:

- Do not mix with other detergents or contaminants
- Avoid over dilution.
- Keep product stored only in original container.
- Keep containers tightly closed when not in use.
- Do not place a wet scoop into container.
- Wash exposed skin and hands thoroughly after handling.

#### WARNINGS:

- Remove Contact lenses, if exposed.
- INGESTION: Do not Ingest.

#### Adverse Events:

- IF SWALLOWED –
  - Harmful, may cause nausea and vomiting.
  - May be irritating to mouth and throat.
- May cause skin irritation, eye damage and allergic skin reactions.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

#### Contents and Setup:

Item No.	Product Specifications
9011996	Brite Shield SAMPLE (1pk)
9011100	Brite Shield (Case of 6)
8101100	Brite Shield Single Jar

#### Materials:

Environmentally friendly Powerful Enzymatic formula.

#### DIRECTIONS FOR USE:

- For daily cleaning and rust protection, place one scoop or one packet of Brite Shield powder and 64oz. (1894ml) of warm water in ultrasonic unit.
- For heavy contaminated instruments, use two scoops of powder.
- For cement/stain removal, place two scoops of Brite Shield powder and 16oz. (473ml) of warm water in ultrasonic unit.
- Stir until dissolved.
- Immerse instruments in solution and clean in ultrasonic unit for 1-5 minutes.
- Rinse instruments thoroughly with water and dry before sterilization.
- For optimal results, change solution per use or daily

#### Disposal:

After use, Dispose as biohazardous medical waste in accordance to facility or institutional guidelines/procedures.

#### Storage:

Store in a cool, dry place away from direct sunlight.

#### Shelf Life:

Brite Shield™ does not have a shelf life.

#### For Further Information:

To obtain SDS or IFU, visit premierdentalco.com or call Premier at 610-239-6000.

#### Symbols:

A full explanation of symbols used on Premier Dental packaging and literature is located at premierdentalco.com/symbol-glossary

Symbols	Symbol Meaning
	Manufacturer
	Batch number
	Date and Country of Manufacture
	Warning, See IFU
	Store in dry cool place
	Avoid direct sunlight
	Medical Device
	Catalog number
	Swiss Union authorized representative
	European Union authorized representative

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