Inspired solutions
for daily dentistry®

Premier®

Premier®

Vac Attak™

High Performance Evacuation System Cleaner

Vac Attak™

High Performance
Evacuation System Cleaner

US/UK

INSTRUCTIONS FOR USE

Intended Use:

VacAttak is intended to be high-performance evacuation system chlorinated cleaner, which is non-foaming, non-corrosive to metals and safe for use with all pumps.

Device Description:

Premier® VacAttak™ is a high-performance evacuation system chlorinated cleaner used in dental practices. VacAttak™ is available in powdered form.

Contraindications:

- Do not use if the container or cap is found broken/damaged
- Do not use on anodized aluminum Instruments or Bur Blocks

PRECAUTIONS:

- Do not mix with other detergents or contaminants
- Avoid over dilution.
- Keep product stored only in original container.
- Keep containers tightly closed when not in use.
- Do not place a wet scoop into container.
- Wash exposed skin and hands thoroughly after handling.

WARNINGS:

- Keep out of reach of Children.
- Avoid Eye and prolonged Skin contact.
- Do not ingest.

Adverse Events:

- IF SWALLOWED –
 - » Harmful, may cause nausea and vomiting.
 - » May be irritating to mouth and throat.
 - » May cause allergic type reaction.
- May cause skin irritation, eye damage and allergic skin reactions.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Contents and Setup:

Item No.	Product Specifications
9011101	Vac Attak (CASE OF 6)
8101111	Vac Attak Single Jar
9011999	Vac Attak SAMPLE (1pk)

Materials:

VacAttak – Powerful organically chlorinated formula and has fresh lemon scent.

DIRECTIONS FOR USE:

- Fill Evacuation sink or other container with 32oz. (946ml) of warm tap water.
- Remove one scoop (8gm) of cleaning powder from container.
- Sprinkle powder into water and allow 30-45 seconds to thoroughly dissolve.
- Flush system with solution.
- Flush system again with tap water after cleaning.

Attention:

Upon initial use, the powerful cleaning action of VacAttak may cause accumulated debris to dislodge. Be sure to check the filter/trap after use for maximum efficiency.

- Highly effective at removing all types of organic debris.
- Use fresh solution daily.

Storage:

Store in a cool, dry place away from direct sunlight.

Shelf Life:

Does not have a shelf life.

Disposal:

After use, Dispose as biohazardous medical waste in accordance to facility or institutional guidelines/procedures.

For Further Information:

To obtain SDS or IFU, visit premierdentalco.com or call Premier at 610-239-6000.

Symbols:

A full explanation of symbols used on Premier Dental packaging and literature is located at premierdentalco.com/symbol-glossary.

Symbols	Symbol Meaning
	Manufacturer
	Batch number
	Date and Country of Manufacture
	Caution: See Instructions for Use
	Keep dry
	Avoid direct sunlight
	Medical Device
	Catalog Number
	Swiss Union authorized representative
	European Union authorized representative

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