Enamel Pro® Varnish

5% Sodium Fluoride Varnish • ACP formula

EN	5% Sodium Fluoride Varnish • ACP formula
FR	Vernis de fluorure de sodium 5 % • Formule ACP
ES	Barniz de fluoruro sódico al 5 % • Fórmula de ACP
KO	5% 플루오르화 나트륨 바니쉬 • ACP 처방서
JP	5%フツ化ナトリウムワニス・ACP 処方

Enamel Pro® Varnish

5% Sodium Fluoride Varnish • ACP formula

Intended Use:

Enamel Pro® Varnish is a fluoride containing preparation for the treatment of dentinal hypersensitivity, and for the reduction of post operative sensitivity. The varnish can be placed on sensitive root surfaces and under temporary restoratives and cements in order to seal exposed dentinal tissue. It can be used as a cavity liner.

Product Description:

Enamel Pro® Varnish is a fluoride varnish which has a strong desensitizing action when applied to dentin surfaces. Enamel Pro Varnish may be used on moist teeth and leaves a film of varnish which sets in the mouth and allow visual control and verification.

Contents:

Supplied in a unit-dose package of 5% sodium fluoride varnish with applicator brush in choice of flavors and container fill (0.25ml or 0.40mL)

Item No.	Product Configurations
9007540	Enamel Pro® Varnish – Strawberries`n cream 0.40ml (Box of 35)
9007541	Enamel Pro® Varnish – Bubblegum 0.40ml (Box of 35)
9007547	Enamel Pro® Varnish – Vanilla Mint 0.40ml (Box of 35)
9007542	Enamel Pro® Varnish – Bubblegum 0.25ml (Box of 35)
9007543	Enamel Pro® Varnish – Strawberries`n cream 0.40ml (Box of 200)
9007544	Enamel Pro® Varnish – Vanilla Mint 0.40ml (Box of 200)
9007545	Enamel Pro® Varnish – Bubblegum 0.40ml (Box of 200)

Materials:

5% NaF Varnish with Amorphous Calcium Phosphate (ACP) formula.

Contraindications:

Do not use Premier Enamel Pro Varnish in cases of:

- Known allergy or hypersensitivity to fluoride or any of the varnish components.
- Patients with known or suspected allergy to tree rosin (colophony).
- Ulcerative gingivitis or stomatitis, as it may cause discomfort when applied.

Precautions:

- Do not use if the package is opened or damaged, prior to use.
- Do not use if the product is discolored, prior to use.
- Brush or applicator is for single use only.
- Avoid contamination of product. Do not return dispensed material to their original container. Dispose of leftover material properly.
- Avoid sunlight or nearby heat sources. Shelf life will be reduced if not stored properly.
- Care should be taken to avoid swallowing the varnish during application.
- Use with caution in young children to minimize ingestion risk.
- Rx only, Prescription device.

WARNINGS:

- Direct ingestion should be avoided. Not for systemic treatment.
- In case of accidental ingestion or allergic reaction, seek immediate medical attention.
- Avoid using in patients with open oral sores or ulcers.
- Avoid applying the varnish on irritated or inflamed gingival tissues.
- Keep away from open flame, highly flammable
- When Enamel Pro Varnish is applied, other fluoride preparations such as fluoride gels should not be administered during the same day. The routine use of fluoride tablets should be interrupted for 2-3 days after treatment.

Adverse Reactions:

Adverse reactions may include but are not limited to:

- Some patients may experience transient tooth sensitivity, toothache, or irritation of oral tissues after application.
- Allergic reactions to fluoride-containing products may occur in some individuals.

- Edematous swelling has been reported only in rare instances in some fluoride varnish products, especially after application to extensive surfaces.

- Dyspnea, although extremely rare, has occurred in asthmatic people.
- Nausea has been reported when extensive applications have been made. If required, varnish film can be removed with thorough brushing.

If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

Dosage: For the treatment of dentin hypersensitivity, in most cases, two applications at several days’ intervals are sufficient for effective treatment. The fluoride content of Enamel Pro Varnish is dosed in such a way that neither acute nor chronic side effects are to be expected if applied according to instructions provided.

DIRECTIONS FOR USE *(Please follow carefully)*

- Before applying the varnish, it is not necessary to have a pre-treatment prophylaxis cleaning and polishing, but make sure the treated area is free of excessive plaque buildup.
- Clean and dry the teeth that will be treated to ensure the fluoride varnish adheres properly.
- Peel off the foil top of the unit-dose container to expose the varnish and applicator brush. Stir varnish with applicator brush. Mix thoroughly as some phase separation of varnish components may occur during storage.
- Apply a very thin and even coat of varnish to the surface being treated using the brush or applicator.
- Enamel Pro Varnish will harden upon contact with saliva and/or water, so the patient can leave immediately after application.
- After the treatment, advise patients to avoid consuming hard food and hot beverages to maximize the fluoride treatment’s effectiveness. Also, instruct them not to brush or floss or consume beverages, mouthwashes, or rinses containing alcohol can accelerate the dissolution of the varnish per the patient post-care instructions. Patient post-care instructions can be printed from www.premierdentalco.com

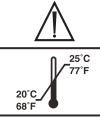
Storage:

- Enamel Pro Varnish can be stored in a temperature range 20°C (68°F) - 25°C (77°F).
- Do not store at temperature over 25°C (77°F).
- Do not freeze.

Shelf Life:

- Enamel Pro Varnish has a shelf life of 2 years (24 months) from the date of manufacturing.
- Do not use after product has expired. See expiry date on the product label.

For Further Information: To obtain an IFU, Patient Post-Care Instructions, and SDS visit www.premierdentalco.com or call Premier at 610-239-6000.

<p>A full explanation of symbols used on Premier Dental packaging and literature is located at premierdentalco.com/symbol-glossary.</p>	
Symbols	Symbol Meaning
REF	Catalog number
	Expiry date
	Date and country of manufacture
	Manufacturer
LOT	Batch number
	Caution: See instructions for use
	The upper and lower limits of temperature of use
	Warning: Flammable liquid and vapor
	Do Not Reuse
Rx ONLY	“CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician

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