



Premier® Enamel Pro® Varnish

5% Sodium Fluoride Varnish • ACP Formula

US / UK

Post-Treatment Patient-Care Instructions

Your dentist/hygienist has applied a therapeutic coating of Enamel Pro® Varnish onto the surface of several of your teeth. You may notice it as thin white film on the tooth surface. The film residue is a temporary condition and should be left undisturbed in order to provide the greatest therapeutic results to the treated areas.

To obtain the maximum benefit from your varnish application, we recommend the following:

- Enamel Pro® Varnish should remain on the teeth.
- Do not brush or floss.
- Eat a soft food diet and avoid hot beverages and products containing alcohol during the treatment period.
- Use of prescribed supplemental fluoride should be interrupted for 2-3 days after treatment (unless otherwise instructed by your dentist or physician).

A thorough brushing and flossing of your teeth will remove any remaining traces of Enamel Pro® Varnish after completion of treatment. Following brushing, your teeth will resume their normal appearance and brightness.